

GUIDEBOOK for EMCHAIR

1. Install the APP

USING cellphone or pad with Android system to install the APP.


2. Turn on the power supply

3. Insert the power line and turn on the power switch



4. Using the APP to control the treatment

5. Set the language

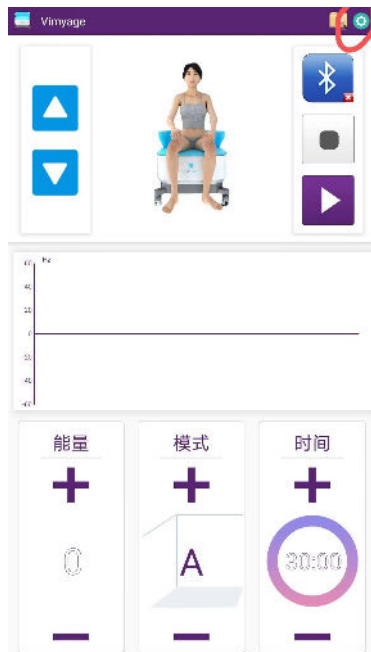


6. Search the bluetooth



7. Connect the bluetooth



8. Blue tooth connect successfully

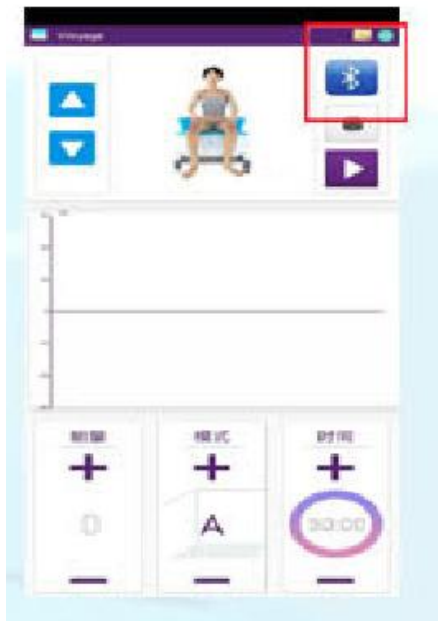

9. Adjust the height for the chair



10. Select working mode



11. Start the treatment

Put ready and start the treatment, Adjust the intensity due to clients feeling



Procedure for Pelvic chair:

1. Confirm with a doctor if you are the right person for the pelvic chair treatment
2. Sit in the middle of the chair at your comfortable position
3. The height of the VIMYAGE chair is adjustable, adjust the height according to the doctor to achieve the correct treatment.
4. Put your feet flat on the floor, and shoulder-width apart. (Different patients has different requirements)
5. Location your knees slightly above the feet at a vertical angle with a little outward rotation.
6. Your pelvic area should be closest to the middle of the chair, maybe you should make some movement to get the best angel.
7. Keep your spine straight and relax. Put your hands on your thighs.
8. The doctor will start the treatment with the Emchair app. At beginning of therapy, the doctors help you gauge positioning. The tapping should be over the perineum (see figure below). Either
9. Adjust forward, back, or towards either side to find the target area.

The treatment has no recovery time, you could back to your normal life immidiately.

CONTRAINDICATIONS

Cardiac pacemakers

implanted defibrillators, implanted neurostimulators

electronic implants

application in the area of growth plate

pulmonary insufficiency

metal implants

drug pumps

application in the head area

hemorrhagic conditions

anticoagulation therapy

application in the heart area

heart disorders

malignant tumor

fever

pregnancy

following recent surgical procedures when muscle contraction may disrupt the healing process

application over menstruating uterus

application over areas of the skin which lack normal sensation