

## Black Carbon Skin Rejuvenation

Choose Treatment head:black carbon tip.

The operation steps are as follows:

1.Spread professional black carbon gel on face, after 20-25 minutes, Gel get well absorbed


2.Laser Setting .for carbon Energy : 450-650 Frequency:3-5 Treatment is around 15-20 mins.

A distance of 8-10 cm is kept between the treatment head and the skin

3. The laser treatment is very moderate; it's s step by step operation. Every 2-4 weeks as one treatment.Result is obviously after one treatment.

4.After treatment ,Use a cotton swab to clean the head with alcohol every time

Treatment process as the following photos:

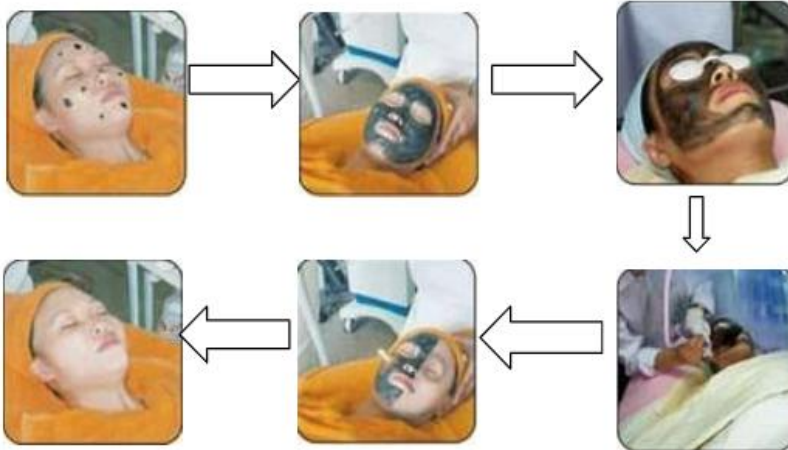

### Skin care after treatment

#### a)Uncomfortable Symptomsin 3-5 days after treatment1.

1.The face will be redness or become swollen,which is a role of heat that will promote collagen regrowth.Pay attention:nevermake ice cooling at this time,just wait for20-60 minutes,they will be gradually disappeared.

2. The skin is very dry,so you have to pay more attention to keep moisture of skin.

3. Somepeople will have a little allergic to carbon powder;you can rub some externalmedicine.

#### b) Cosmetic and skin care products

1. Do not clean face with too hot water and irritant clean products;

2. Choose some moisture lotion and sun protection lotion for skin care;

3. Use some moisture essence,such as Hyaluronic Acid,B5 for skin repair

#### c) Daily Life

Do not to go to High temperature places in 3-5 days after treatment,such as sauna,Hots pring,sunbathing