

10 in 1 vertical water dermabrasion machine



User manual

First of all, let's express our heartfelt thanks to the customers who bought our instruments. This manual is written for customers to better use the instrument, hope the company's products and services to bring you respect.

Attention:

The product must be used in strict accordance with the use methods in this manual. All consequences caused by improper use of methods other than the instructions shall be borne by the company, and the company shall not bear any responsibility. The Company makes no warranties of any kind, including (but not limited to) implied warranties of affordability and suitability for a particular purpose. The Company shall not be liable for accidental or consequential damages caused by errors contained in this document or by the provision, actual performance and use of this manual.

This manual contains proprietary data protected by patent law. All rights reserved. No part of this manual may be photographed, reproduced, reproduced or translated into other languages without the written consent of the Company.

The content contained in the manual can be changed without notice to the user.

In this manual

This user manual contains all the general information about the instrument. This is the starting point for new users to start using the instrument, because it gives a general introduction to the system, describes how it works, introduces the instrument installation and various operating methods. You can find the required data through the catalog. Re-examination of the personnel reading.

This manual is intended for those who are familiar with the aesthetic knowledge of the instrument and who have experience in the use of aesthetic equipment. If you have any questions, please contact the instrument salesperson.

Please use it safely

The following symbols are used in the instrument and instructions. In order to use the instrument correctly, prevent injury to operators and patients, and protect the instrument from damage to avoid property damage, please pay attention to the types and meanings of the following symbols.

ASM:	Indicates that there is a risk of personnel damage and instrument failure due to operational errors	A	Mark table loose items
	Please refer to the instructions		The protection type of the equipment is Type B.
Note	Remind yourself to highlight important information		

1. Introduction and Installation Instructions of the Instrument



1.1.1 Please check the goods before signing for acceptance. Carefully remove the instrument and accessories from the packing box, check for any mechanical or visual damage, and photograph and reject any damage.

1.1.2 Store the packing material for later transportation or preservation (keep the outer packing for one year). Count the accessories according to the packing list.

1.1.3 The appearance of the product is different. The above figure is for reference only, and the specific object shall be taken as the reference.

1.1.4 Any technical improvement of the product will be included in the new manual without notice

Note:


Contact our sales department or agent immediately if you have any questions.

1.2.1 The instrument shall be placed in a clean, ventilated room with a temperature of 5 °C-40 °C and no more than 80% to avoid direct sunlight. Ensure that the vents are not blocked and that there is at least 5 cm clearance around the instrument.

Remember not to place items on the instrument. Even if they are temporary, they are not allowed. Please do not wrap fresh film on the instrument, which may cause the paint to fall off or other appearance problems, which belong to artificial damage. We do not give any return.

1.2.1 Insert the accessories into the corresponding interfaces according to the text and picture information prompts above the output interfaces on both sides of the instrument. Then place the fitting neatly on the fitting frame.

2, Instrument interface operation and description

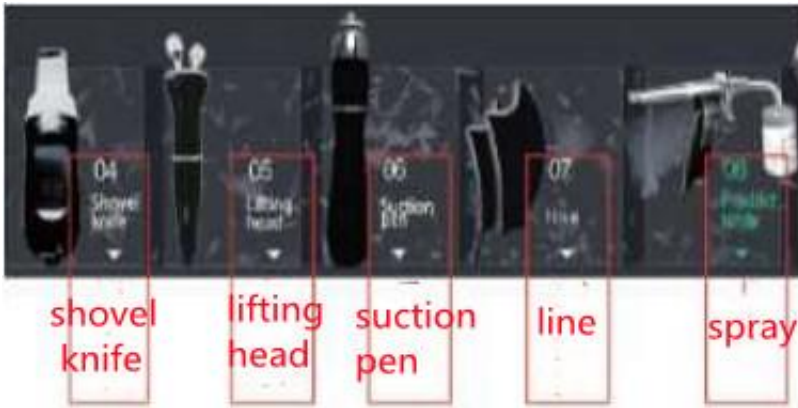

Operation method: Slide accessory ascaris bar, g click accessory fflA work binding

Note:

When the instrument is powered on, it is not allowed to pull out the probe or replace the probe at will, nor move the probe at will, resulting in circuit damage, which belongs to artificial damage.

Note:

The intensity and time are usually adjusted slowly from low to high. The comfort level of the customer shall prevail. The mode shall be selected according to the requirements of the customer.

Xianyì:

In case of discomfort, discontinue use immediately and seek medical advice in a timely manner.

Attention:

Pregnant women are forbidden to operate and use the instrument. After the

instrument is turned on, keep away from the instrument.

3. Precautions

3.1 Precautions for use place

Warning:

- 1) Do not use and connect sockets to avoid fire;
- 2) When replacing the fuse, turn off the power switch to avoid fire and electric shock;
- 3) 220 V+22 V 50HZ±1HZ power supply and effectively grounded socket must be used;
- 4) Danger of fire or electric shock with damaged or processed power cord
- 5) Do not overbend or twist the power cord to avoid fire or electric shock.
- 6) If the instrument is loose, dropped or damaged, please contact the company in time;
- 7) Do not use the instrument in unstable environments such as tilt, vibration and impact;
- 8) There is no water or sundries inside the instrument. In case of any sundries, please contact the company in time;
- 9) The power plug must be unplugged before the instrument moves;
- 10) When the instrument moves, vibration shall be avoided as far as possible;
- 11) The instrument shall be placed in a stable place, and a space of at least 5 cm shall be reserved around it to ensure air circulation and facilitate heat dissipation of the instrument;
- 12) The instrument shall not be covered with anything to prevent the vent from blocking;
- 13) Avoid using the instrument in the following environments: Sunlight exposure; High humidity environment; Near-water environment; Places of vibration and inclination; Strong magnetic field environment; Electromagnetic wave and impulse voltage; Places where chemicals are stored and places where corrosive gases are found;
- 14) The instrument shall not be placed near radios, television sets, photocopiers, fax machines, etc.;
- 15) Do not use with other instruments (such as microwave) to avoid interference and misoperation.
- 16) When the instrument is turned on, do not unplug any probe of the instrument, otherwise the instrument will be damaged directly.

3.2 Precautions before use

Warning:

- 1) Be sure to read the instructions carefully before starting the machine. The operator must be familiar with the operating instructions and operation methods, and the instrument must be managed by special person;
- 2) The operation parameters shall be set under the guidance of professional personnel;

Note:

- 1) Confirm whether the instrument is in normal operation state before use;
- 2) Confirm whether the grounding wire is correctly connected;
- 3) Make sure all wires are properly connected and safe;

4) When the instruments are used together, be sure to use them correctly. Otherwise, there may be danger and great attention must be paid.

Police.

1) In case of abnormality during operation, the operator shall immediately handle the operation handle.

Leave the operator and press the stop switch to stop the use of the instrument;

Public idea:

2) During the operation of the instrument, continuously observe the condition of the beauty guest and the instrument, and pay attention to whether abnormal conditions occur at any time:

3) The instrument shall be correctly used by skilled operating technicians. Personnel other than operating technicians shall not use the instrument.

Precautions after use in 3.4

△a meaning:

1) After the operation is completed, the control handle is placed back into the control handle hanger.

2) After using the operating handle, clean the probe for next use.

3. 5Precautions during failure, storage and inspection

Oath:

△) Please do not use the instrument box under abnormal conditions. In case of inoperability due to burning smoke, continue

There is a risk of fire and electric shock when used. Press the stop button quickly,

Turn off the power supply, unplug the power supply, and contact the company or distributor immediately.

2) No personnel other than technicians shall be allowed to repair or modify the instrument. The inside of this instrument must not be opened.

Zhu En-fu and Du Shi must be at first;

4) Do not use cleaning tools that damage the protective coating of the instrument, such as brush, broom, etc.; When the instrument is not in use for more than 2 weeks, the instrument must be carefully checked before reuse.

4. Contraindications

Diseases such as heart disease, gastric ulcer, severe gastric disease, duodenal ulcer, hypertension, diabetes, etc.

92. Women, breast-feeding women, abdominal surgery less than three months, menstrual period is prohibited.

53. A person whose body temperature is more than 38 degrees.

◎ 4.A person vulnerable to a polluting disease.

◎5.An acute illness.

◎6.A heart attack.

97. extreme hypertension, cerebral infarction and other serious patients.

88. when the symptoms of contact dermatitis and atopic dermatitis are serious.

◎9.Hemangioma, vesicular rash, follicular sores, lupus erythematosus to body,

systemic sclerosis, Addison's disease.

@10.Hemorrhagic disease or trauma.

911. A woman with an intrauterine device.

912. Pregnant women are prohibited to operate and use, the instrument should be away from the instrument after power-on, if there is a problem: The consequences of self-responsibility.

Please use this instrument under our guidance.

54. Allergic skin.

015. Wound skin.

. 16.Cannot wear metal accessories.

@17.Minors may not be used.

5. Maintenance and repair

5. 1Maintenance

5.1.1 Daily inspection management personnel must inspect the instruments and components.

a) Whether the power socket is reliably connected to the ground, and the power socket is not replaced.

b) Whether the electric wire of the power supply is deformed, broken or broken: By visual inspection, whether the electric wire is deteriorated or deformed, if any, it may cause fire due to electric leakage. Replace the power cord immediately.

c) Whether the control handle is abnormal or broken: The output line is deteriorated or deformed through visual inspection. If yes, please replace it immediately.

d) Please check if the prompt content before using the instrument is correct, and stop using if it is incorrect.

5 1 2 Cleaning

When cleaning the rattan, the dust must be wiped with a dry soft cloth and the loot with a soft wet cloth. Before cleaning the instrument, turn off the power switch and disconnect the power cord.

Warning:

To prevent electric shock when replacing the fuse, turn off the power supply and unplug the power plug.

5.1.3 If the instrument needs to be cleaned for a long time (more than one year), it should be carried out by professional maintenance personnel.

5.1. When the 4 ft device is not used, it shall be placed in a ventilated and dry place, and the instrument shall be covered with a dust cover.

5. 2Maintenance

5.2.1 After purchasing the instrument, you need to fill in the warranty card and send it to the manufacturer or agent to have the warranty effect; From the date of instrument production, it belongs to the free warranty of the company's product quality problem host for one year. The warranty here is limited to the instrument main unit, excluding the consumables to be replaced regularly such as accessories).

5.2.2 After the warranty period, please contact the manufacturer directly to provide service coverage.

5.2.3 The machine must be repaired with our customer service department or dealer contact, do not repair without permission, self-unpacking, our company does not

warranty.

7. Replacement of accessories

Accessories must be purchased from us when they need to be replaced.

8. Storage and transportation

8.1 Storage: The instrument shall be stored in the original packing box and placed in a clean room with good ventilation. The packing box shall be elevated and the ambient temperature shall be $-10^{\circ}\text{C}\sim 40^{\circ}\text{C}$. The relative humidity is not more than 93%. No harmful gas, flammable and explosive substances or corrosive substances are allowed.

8.2 Transportation: Instruments under packing conditions are suitable for road, railway, aviation and waterway transportation. During loading, unloading and transportation, they shall be protected from train vibration and shock, and shall not be subject to moisture. They shall not be mixed with inflammable and corrosive substances for loading and transportation. Specific requirements shall be specified in the order contract.

9. Lifetime

1 The warranty period of this instrument is 1 year.(Except for artificial damage)

2 The warranty period of accessories is 3 months,(Except for artificial damage)

3 The instrument beyond the service life shall be inspected and approved by the engineer for further use, and the performance of the instrument shall be inspected by the engineer at regular intervals during subsequent use.

4. The machine shall be responsible for repair and update if the warranty period of one year (except for vulnerable parts) of other non-warranty scope of the host machine has expired since the date of instrument production (by bill), and the repair fee shall be charged appropriately;

5. No free warranty service is provided for the following personal reasons;

- ① Malfunction caused by dismantling and refitting the product without authorization;
- ② Failure caused by accidental beating or falling;
- ③ Failure due to reasonable maintenance;
- ④ Failure to follow the correct instructions in the instruction manual.

Instrument after-sales card

(This list is attached to the repair instrument)

Product Name:	Purchase date:	
Customer Name:	Tel:	
Delivery date:	Date of delivery:	
Fix return address:		
Instrument fault description:		
Year Month Day		
Accessory Name	Unit	Quantity

Thank you for purchasing our instrument. If there is any need for maintenance, please fill in the maintenance form in detail and send it together with the returned instrument. The information you provide is very important to us. We will repair the instrument as soon as we receive it.