



An instruction manual

Gold radio frequency micro crystal

Note: read the instructions, operation items and precautions before operation


What is the gold radio frequency microcrystal?

The gold radio frequency microcrystal is the ingenious combination of micro crystal and radio frequency. The two word of "gold" originates from the microcrystalline gold coating and the coating is also golden yellow. At the time of treatment, the doctor according to the crystal position on Problems and treatment, adjust the depth of penetration and microcrystalline radiofrequency energy, then in the electronic control system, at the same time, dozens of insulating ceramics penetrate the skin quickly, from the micro crystal tip radiofrequency energy, then quickly exit, so the cycle until the treatment is finished, finally apply cosmetic ingredients.


Action mechanism

- 1, micro crystal mechanical stimulation, coupled with the biological effects of radio frequency and thermal stimulation, together to stimulate the skin's self repair system, to promote metabolism, improve microcirculation, the initiation of collagen and other new and rearrangement;
- 2, microcrystalline penetration, open the skin quickly absorption channel, easy to cosmetic ingredients into the skin;
- 3, the radio frequency energy emitted by the micro crystal tip can selectively destroy the hair follicles, sebaceous glands and sweat glands, inhibit inflammatory reaction and activate the anti-inflammatory and repair system.

Gold radio frequency microcrystalline vs dot matrix laser difference

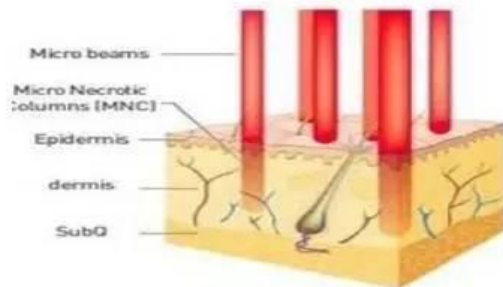

fractional laser

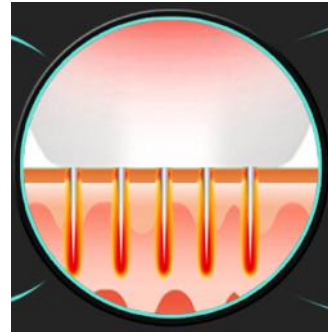

Gold radio frequency micro crystal

1. the high frequency crystal head is formed by the crystal depth of physical properties, forming heating and solidification fields.
2. gold RF microcrystalline target depth and required energy can be finely regulated.
3. laser scattering, but it is difficult to reach the target depth (target 2mm), and the absorption of energy within the skin tissue can not be correctly predicted, often occur side effects such as hormonal disturbances.
- 4., heating through the epidermis, and a longer rest, and it is difficult to calm.
5. lattice laser a course of care time must be reached (3~5 times), while relative to laser RF-Needle is according to the circumstances may need (1~3 times) care.

Instructions

Instrument front :



Instrument back :



Handle and crystal head:



Once crystal: first with saline before use: disinfection

disinfection can place the probe in a small bowl, alcohol can not be more than the probe before operation, the probe must not the remaining water, to avoid any damage to the machine machine (such as two times of any problems had nothing to do with this company)

10p, 25p, 64p nanocrystals



Operation interface:


Main interface :




F facial operating interface:

Manual—— Manual mode

Auto—— Automatic mode

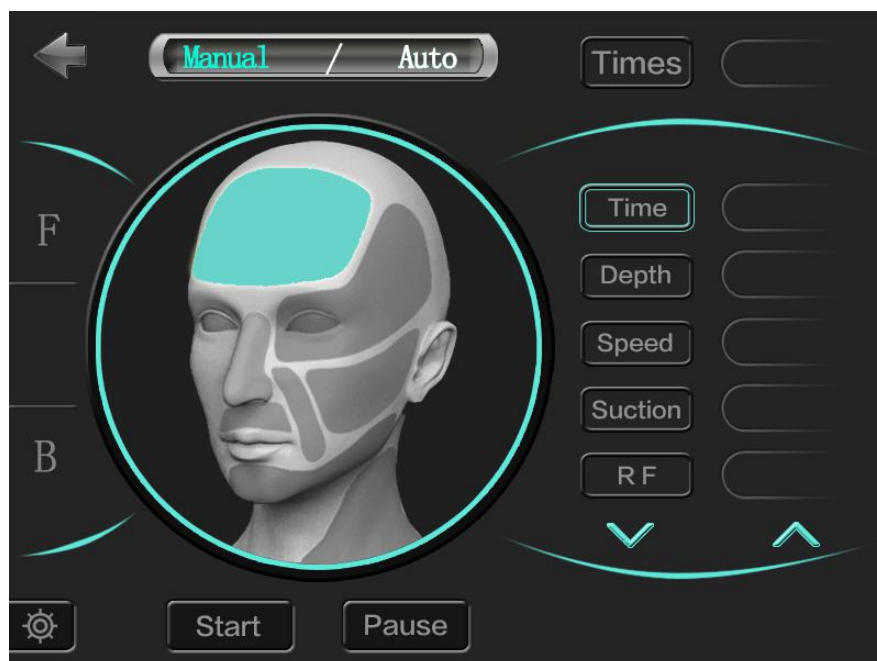

B belly operation interface:

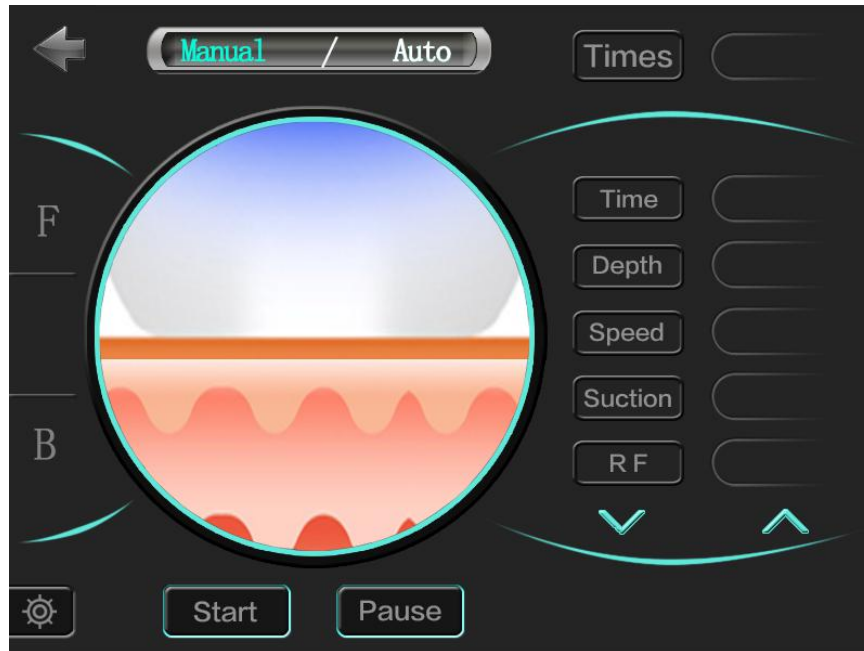

Background interface:



RF energy
regulation
(not easily)

Reference parameters for therapeutic sites:

Therapeutic site	depth	Speed, i.e., energy output time	suction	RF size
forehead	0.5-1.0mm	0.2-0.3	1-2	60-110W
Face	1.0-1.5mm	0.2-0.3	1-2	60-100W
Eye	0.5-1.0mm	0.2-0.3	1-2	60-100W
chin	0.5-1.2mm	0.2-0.3	1-2	60-100W
nose	0.5-1.0mm	0.2-0.3	1-2	60-100W
neck	1.0-1.5mm	0.2-0.3	1-2	60-100W
Pox pits	1.0-2.0mm	0.2-0.3	1-2	60-100W
scar	1.0-2.0mm	0.2-0.3	1-2	60-120W
belly	1.0-3.0mm	0.2-0.3	0-1	60-120W

Nursing methods:

1. cleansing - before cleansing, cleanse the skin and deep cleanse.
2. - to promote circulation of nursing Ma parts of the painting paste, and plastic film and improve sisal paste effect. (about 30 minutes)
3. sedative care - apply a mask or cold mask for sedation.
4. gold RF microcrystalline care - operation time according to different parts, but generally require 15 to 20 minutes, the operation only needs 1 to 2 times can not remember, can be repeated many times, effect of scab off time..

5. nursing time: 3 to 4 weeks, once the scab care, a course of treatment 3 to 5 times, 1 times a month.

6. repair nursing: after care, 24 hours of care parts do not touch the water.

The following conditions may occur after treatment:

Redness, swelling, fever.

1 days later: fever, redness, and according to the constitution, there is a slight swelling.

2 to 3 days later: skin, erythema, or pox.

3 to 4 days later: peeling and shedding naturally.

Matters needing attention:

After the care, in order to repair the skin, and sooner or later apply moisturizing and repair products. After the scab, and need to fall off naturally, hand off to avoid.

Sunscreen should be applied to avoid UV light. Avoid strenuous exercise, swimming, sauna and so on for a short time. Alcohol is prohibited for a short period of time.

Alcohol can cause redness to last for a long time and is not conducive to repair.

Therefore, it is forbidden

Hidden in the skin of the aging crisis

20 years old hidden in the corner of the eye crisis



In all the wrinkles, crow's feet start to grow around 20. When you look in the mirror face will find some small fine lines. The skin of

the eye is the thinnest and most fragile on the face, so pay attention to eye care

The 30 year old wrinkles, apple muscle, Temple



Wrinkles began to appear to be a little late, in addition, also influence muscle sagging wrinkles. Similarly, there are also drooping Apple muscle, oh, as the face of the most smooth and soft piece, the apple muscle like the muscles stored a lot of fat, so it is particularly easy to be affected by gravity.

The part of the muscle in the temple is thin, which gradually causes the shape of the bones to appear and become uneven

40 year old legal pattern + eyelid and chin droop



By 40 years of age, muscle relaxation is more pronounced, mainly in the eyelids and jaw two parts that you do not notice. Relaxation of muscle and fat drop let mandibular downward raised meat, positive is not obvious, to face minute exposure. Eyelid skin is the same, especially the eyelid, ptosis will be more significant. The statute line is the last part of the trilogy of aging

50 year old lip wrinkles




At the age of 50, under the nose to reduce fat, flabby skin excess extrusion to form longitudinal stria lips, once deep lip wrinkles, your skin has entered the final period of aging, no matter how the use of cosmetics is not stop the modification

Gold microcrystalline - your wrinkle Terminator

What is the main problem of gold microcrystal?

- 1, wrinkles, crow's feet wrinkles wrinkles neck wrinkles.
- 2, facial rejuvenation, compact lifting
- 3, Stretch marks of stretch marks
- 4, large pores
- 5, dim skin
- 6, red blood
- 7, skin whitening
- 8, acne scars
- 9, sunken scar
- 10 、 regular skin care and maintenance

How does the gold microcrystal work?

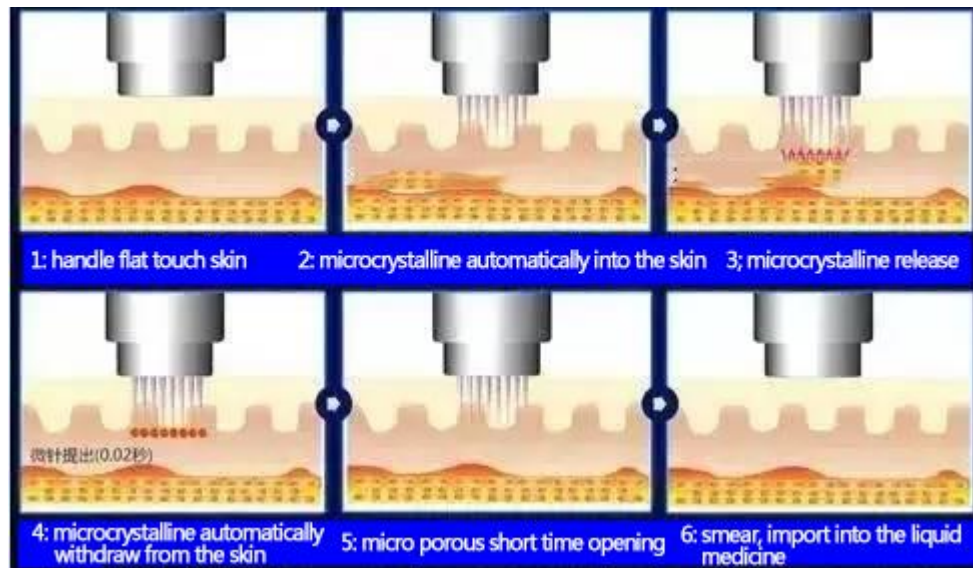

Part1

Lattice physical microcrystal——Firming lifting: break through the skin barrier, and target the deep layer accurately.

Part2

High energy constant RF——Full and wrinkle free: radio frequency energy stimulates the synthesis of collagen, elastin, and hyaluronic acid, and the skin becomes Fuller, hydrated, Q, and free of wrinkles.

Part3

Transdermal delivery——Bright, white, transparent, tender and tender: along the channels created by microcrystalline, personalized therapy, drug introduction, the effect is better

Gold microcrystalline, what's the effect?





Why is gold microcrystal so popular

Reason one:

New treatments double effect

RF technology, firming microdermabrasion technology, break through the traditional single treatment, the dermis directly emit RF energy at the same time, according to the different degree of adjustment of microcrystalline penetration depth and the size of RF energy, double effect.

Reason two :

Not scab, colorless, sink, perfect effect

Gold microcrystalline using unique design of insulating microcrystalline, to avoid stimulating the epidermis, the crystal tip to the treatment depth, release energy, no thermal damage to epidermal tissue, postoperative will not appear scab, color and other adverse reactions, the effect is perfect.

Reason three:

The curative effect is remarkable and the effect is maintained for 3~6 years

The precise role of different depth of target tissue, real-time monitoring and intelligent impedance, crystal of wrinkles, relaxation, acne, scars and other symptoms, personalized settings, make the skin rapid repair to the young and healthy state, the longer the better, a treatment lasts for 3-6 years.

Gold radio frequency microcrystal contraindication:

Skin infection or skin diseases (except acne), hemorrhagic diseases, taking anti clotting drugs, pregnancy, physical scars, wearing a pacemaker (heart rate devices and other electronic equipment, the body) or metal stent implants, skin malignant tumor, no treatment recommendations.

Technical specifications:

output frequency	2-4M
output power	10-200W
Therapeutic probe	10p 25p 64p
input voltage	AC 110V~220V~50Hz/60Hz

parts	Free warranty for half a year
Host	Free warranty for one year

