
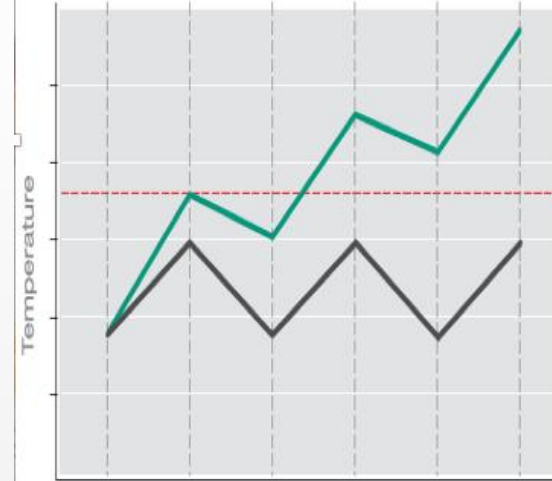


# How Does IPL work?

It is a revolutionary approach, rather than bombarding your skin with high amounts of damaging energy. IPL fires multiple shots but at low Joules, in doing so it gently heats the hair follicle to the heat needed and the most you will feel is warmth and a tingling sensation, some clients comparing it to a warm massage. IPL also utilizes In-Motion technology, where the hand piece is always in motion over the skin.

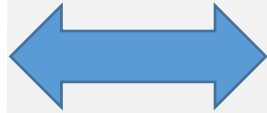


Injury threshold

What it does:

Multiple-Sequential Pulsing allows for cooling between pulses, to safely remove hair on darker skin and reduce the chance of skin damage in vascular and pigmented lesion treatments.

— Epidermis — Lesion

## HAIR REMOVAL

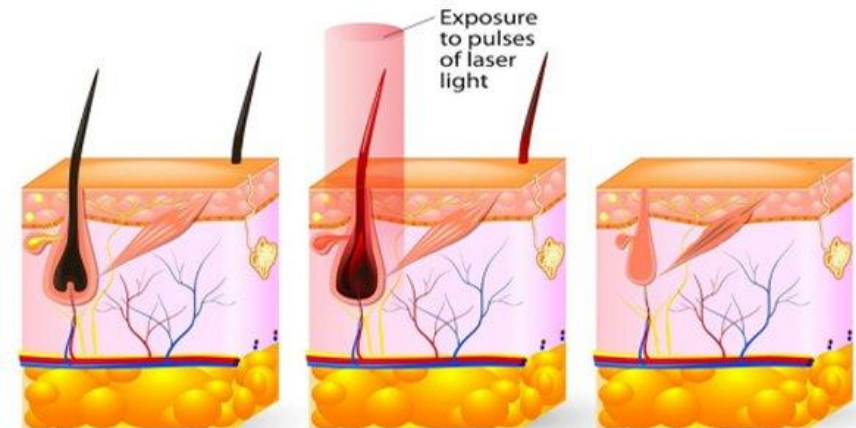

# Why Choose It?



As the new 3rd Generation of Laser & SHR Hair Removal, **It** combines two advanced technologies: **MPT™ (Medical Pulse Technology)** and **HS-PCT™ (High-Speed Pre-Chilled Technology)**.

**(1) AFT (Advanced Fluorescence Technology)** = Optimal Spectrums for a board range of Therapeutic Combinations

## AFT 650nm - Optimal for Hair Removal


**Step 1** - AFT filter converts unused UV light into visible red light

**Step 2** - Additional red light enhances treatment efficacy

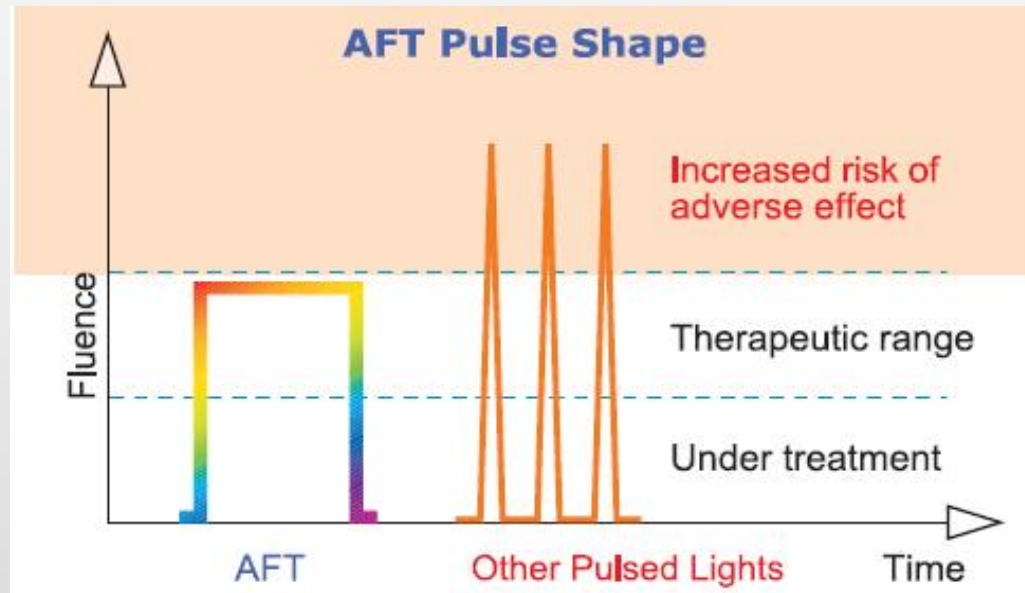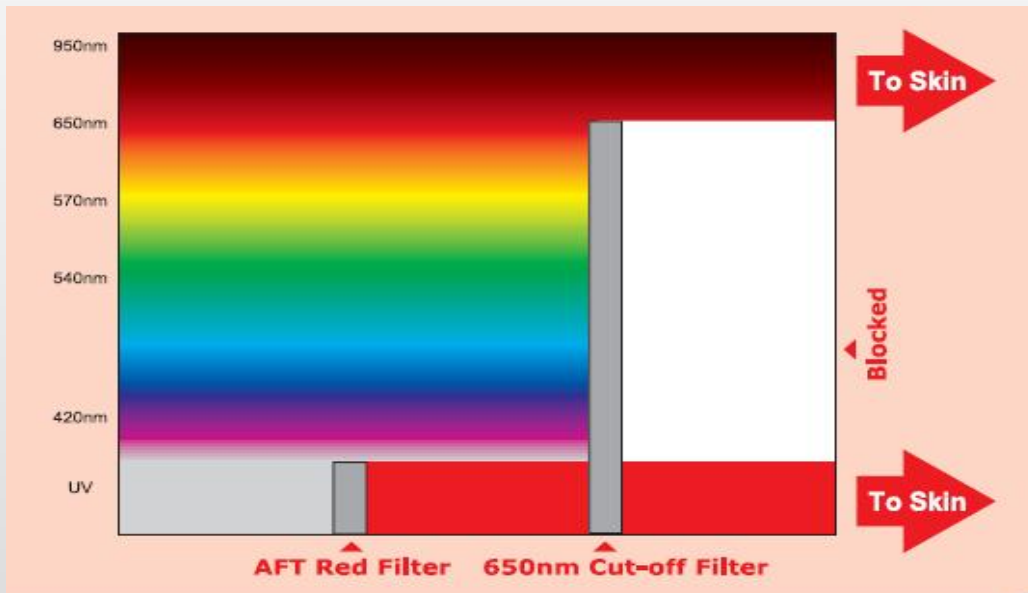

**(2) FCA (Fluence Calibration Automatically):** Combining the level of Customized Power Supply + Brand-new Japanese Capacitors +UK Superior Xenon Lamp + Pure Sapphire Crystal + Intelligent Software, gradually heating the dermis to 45 °C via a pulsed stacking method, creating strong enough energy by single pulse to distribute the efficient light of 650-950nm to reach the target tissue!

### **Powerful & Durable Xenon Lamp from Germany**

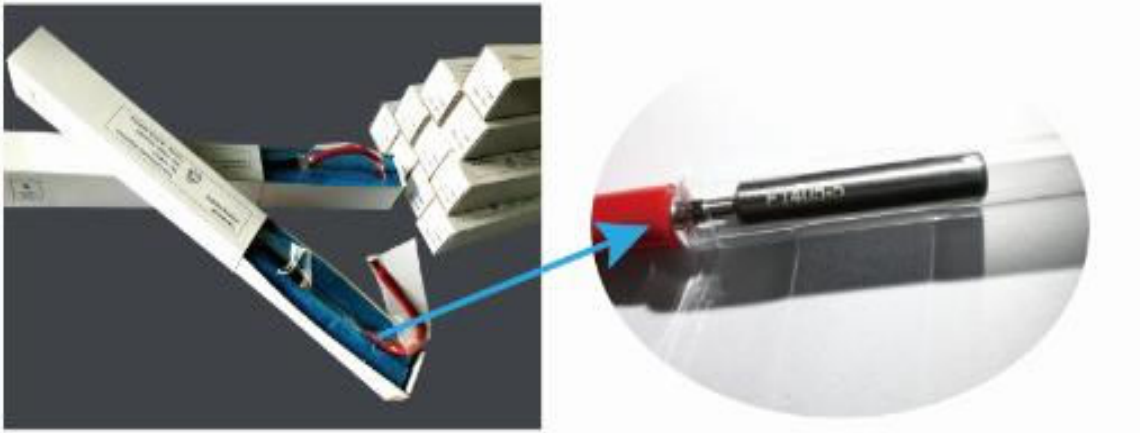

## **What Conditions does IPL Photorejuvenation treat?**

- Young patients with milder degrees of sun damage or early aging changes
- Rejuvenation of skin of the face, neck, chest, shoulders, arms or hands
- Pigment Changes/Sun Damage or Aging Skin
- Ruddy Skin Appearances ("Dyschromia") - rosacea, redness of the chest and neck
- Enlarged Pores
- Fine Lines
- Telangiectasia (broken capillaries)
- Unwanted Freckles
- Hyperpigmentation or brown spots

## **What to do Before your IPL Treatment:**

- Avoid tanning and excessive sun exposure 4 weeks prior to your treatment.
- The treatment areas should be cleansed and all makeup or creams removed.
- Each treatment takes approximately 15 to 45 minutes depending on the size of the area treated

**(1) Handle piece with good cooling !**





### (3) Intuitive and Intelligent Operation Interface

**E-LIGHT**

Present physical therapy mode: Hair Removal

Physical therapy record: 1, 2, 3

Secondary pulse width: 12.0, 12.0, 12.0

First pulse width: 12.0, 12.0, 12.0

pulse interval: 10, 10 ms

Number of pulse: 200 ms, 3, 2 S

RF time: 200 ms, 3, 2 S

Pulse interval: 616, 18

Total flash: 616, 18

Flash this time: 616, 18

Status bar: 0.00 C°

Buttons: Standby, Ready

Adjustments: IPL (10 J/CM<sup>2</sup>), RF (2 J/CM<sup>3</sup>)

Refrigeration: 4 snowflake icons

Back: Green arrow icon

**SHR**

Pulse width adjustment: 1.0 ms

Frequency adjustment: 3 Hz

Flash this time: 616, 18

Total flash: 616, 18

Status bar: 29.00 C°




Buttons: Standby, Ready

Adjustment: Energy Adjustment (5 J/CM<sup>2</sup>)

Refrigeration: 4 snowflake icons

Back: Green arrow icon

### (3) Intuitive and Intelligent Operation Interface (16 Languages in one system)

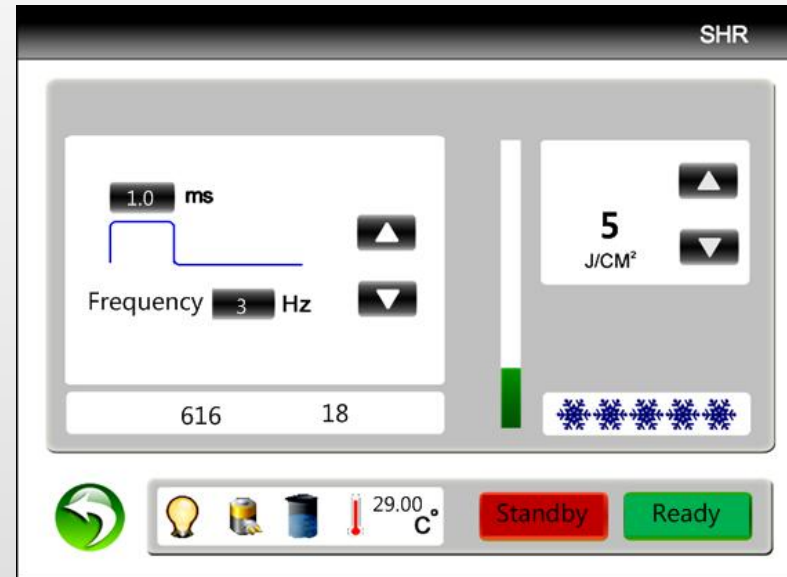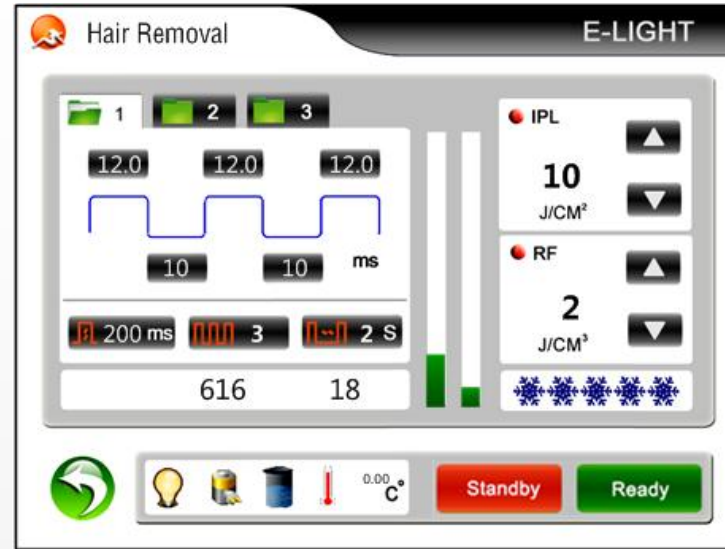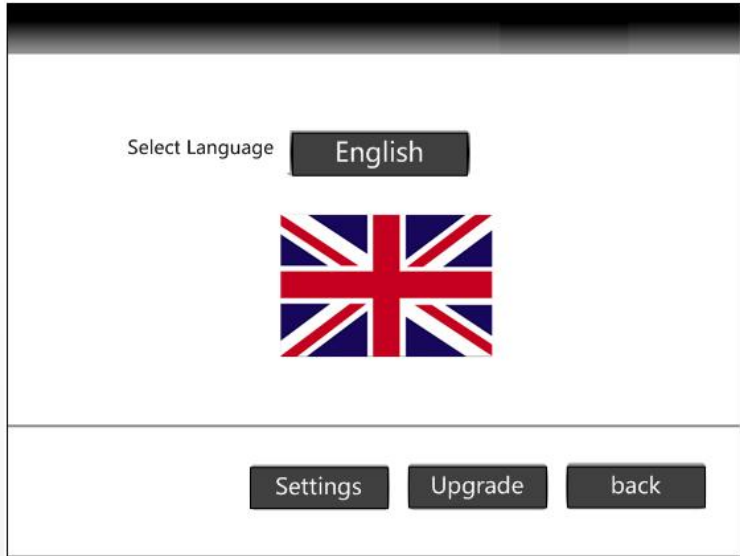


## (1) High Quality Power supply can support machine with big energy !



## (2) HS-PCT™ (High-Speed Pre-Chilled Technology).

- The high fluence Pre-Chilled Technology applies a cooling mechanism to chill the skin before and during treatment, increasing treatment efficacy and patient comfort.
- 7 Liters Water Tank + 4\*Radiators + 20\*Fans + 60W Semi-conductors ensure the great refrigerating capacity of handle, and makes the contact skin still  $-2^{\circ}\text{C} \sim 0^{\circ}\text{C}$  even after a long working time.

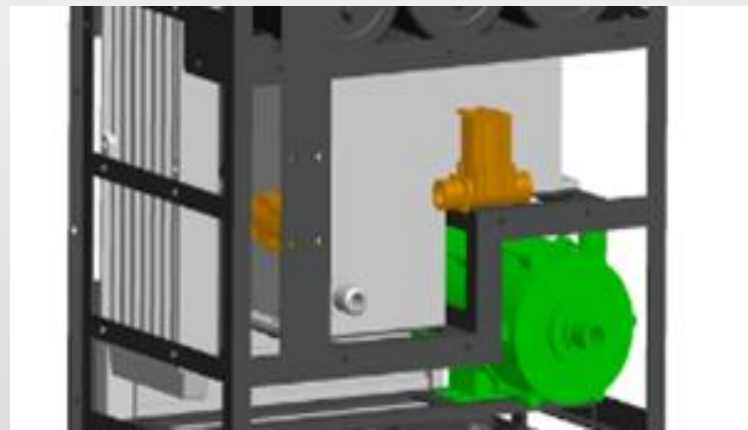

# Technical Specifications

Illuminant	IPL	Spectral Range	430/480/530/560/640-1200nm(E-LIGHT) 640-1200nm (SHR)
Illuminance Area	15*50mm(E-LIGHT) 10*40mm(SHR)	IPL Energy	1-50J/cm <sup>2</sup>
Adjustable pulse width	2-99ms(E-LIGHT) 0.1-20ms(SHR)	SHR Frequency	1-10Hz
Display Screen	8.0- itch color touch screen	Cooling system	Water cooling + semiconductor refrigeration, air cooling
Total Power	2000W	Voltage	AC220V / AC110V Alternative
weight	50kg	package size	



# Clinic result

SHR & HR



SSR & SR



AC



VL/PL



ST



## What to do After your IPL Treatment:

- A sunburn sensation is common. Cold packs may be used for ten minutes after your treatment.
- The skin may appear pink and flushed. This usually resolves after several hours.
- Age spots and melasma patches will darken over several days and either exfoliate off over 7 to 10 days or fade over several weeks.
- Striping is common in areas treated off the face, like the arms and chest, and require follow up treatment to even out the results.
- Gentle cleansing and sun protection are important while the skin is sensitive.
- It is very important to avoid tanning 4 weeks before and 4 to 6 weeks after your treatments.
- In general, there is little downtime with IPL. Occasional bruising and swelling may occur but is temporary and resolves over 3 to 7 days. You can resume normal activities immediately after your treatment.
- Gradual improvement will be noted over your course of treatments.

## **IPL Treatment Menu Pricing**

### **IPL Photofacials Pricing**

Full Face: \$200 FaceandChest: \$300 Face,Neck,andChest: \$380 Hands: \$140

### **IPL Pigmentation Pricing**

Such as redness, freckles and post-trauma pigmentation such as acne scars and age spots.

Nose: \$40

Cheeks: \$70

Quarter Face: \$83

HalfFace: \$111

FullFace: \$139

Neck: \$98

Decolletage(Chest): \$139 Face+Neck: \$213 Face+Neck+Decolletage: \$279 Hands: \$139

Shoulders: \$139

LowerArms: \$174

FullArms: \$347

Lower Legs: \$272

FullLegs: \$544

UpperBack: \$376

FullBack: \$531

### **IPL Hair Removal Pricing (per treatment)**

Lip: \$59

Chin:\$69

Jaw:\$79 Sideburns:\$99

Full Face: \$250

Ears: \$49

Brow Middle: \$30 Brow Forehead: \$49 Between Breast: \$49 Areola: \$59

Lower Navel Below Belt Line + Belly Line: \$79 Bikini Line Traditional 2 Fingers From Crease: \$139 Bikini line Extended with Inner Thighs: \$169 Brazilian Everything Front + Back: \$225

Buttocks (Glutes): \$69 Brazilian Backside: \$69 Inner Thighs: \$69 Underarms: \$119 Upper Arms: \$129 Lower Arms: \$129

Full Arms: \$199 Hands: \$99

Feet: \$69

Back of Neck: \$69 Lower Back: \$69 Upper Legs: \$219 Lower Legs: \$219 Full Legs: \$399