

Magic Pot RF Face Care and Body Slimming machine

USER MANUAL



Warning

Dear user:

Thanks for using our company's products, in order to fully make use of the products, we sincerely suggests follow:

- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

This product does not belong to medical, please don't use for disease treatment!

Parts Introduction



- 1.Touchscreen
- 2.Function selecting knob & Confirming button
- 3.Accessories frame
- 4.Multi-RF photon Vacuum liposuction handle jack
- 5.Filter cup
6. Power supply line socket
- 7.General power switch

Pull out protective cover with tweezers or other tools
Wash or change the filter cotton inside



Accessories



Multi-RF photon Vacuum liposuction handle connect cable



Large Multi-RF photon Vacuum liposuction head



Middle Multi-RF photon Vacuum liposuction head





Small Multi-RF photon Vacuum liposuction head

Any part of instrument made of PMMA material (for example: accessories shelf, panel, transparent handles,etc.). Do not use alcohol or corrosive solvent to wipe clean.

Basal Operation

1. Put accessories well and connect them to the host.

(1) Insert the connect cable of Multi-RF photon Vacuum liposuction handle of into [4] jack. Pull the pull-down switch, according to arrow direction. turn rotating ring to lock direction, If need to pull out, pull the pull-downen switch rotate the rotating ring to “UNLOCK” direction. then pull it out.

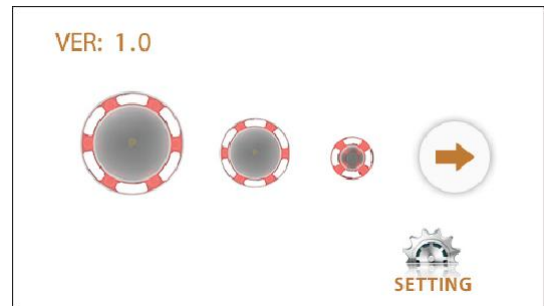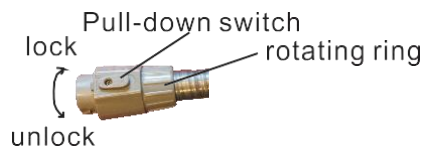

(Fig 1)

(2) Select appropriate Multi-RF photon Vacuum liposuction head, Insert it into the other end of Multi-RF photon Vacuum liposuction handle connect cable. The method is the same.

2. Insert the plug of Power supply line into [6] socket, turn on Power switch [7], put into the system default password “000000” to enter interface as (Fig 1).

3. In the interface as (Fig 1) shows:

(1) Click on “SET” to to select output language. Click on “advanced setting” in this interface to enter engineering debug interface. **NOTICE: “Advanced setting” is only for manufacturer, Users do not use.**

(2) Click on “→” to enter treatment interface as (Fig 2) shows.


4. This machine can click the keys on the touchscreen to set the working parameters, it also can use [2] Function selecting knob & Confirming button to set. Turn the knob when you choose the function, the icon “↖” move to the right place, then press the knob [2] to confirm.

5. This use’s manual use touchscreen for example.

Vacuum Lipo-suction


Operating steps

1. Click on “→” in the interface as (Fig 1) to enter interface as (Fig 2) to do treatment.
2. Ask the guest to have a bath or clean the nursing part, then smear the essential oil, the amount depends on moving flexibility of the Multi-RF photon Vacuum liposuction head.
3. According the treated parts to choose appropriate liposuction head. Large Multi-RF photon Vacuum liposuction head is suit for abdomen, back, thigh etc. fatty parts. Middle Multi-RF photon Vacuum liposuction head is suit for crus, arm etc. Small Multi-RF photon Vacuum liposuction head is suit for facial.

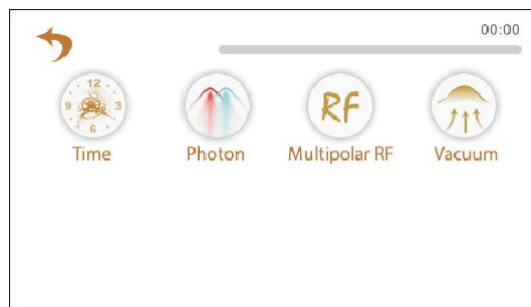


(Fig 2)

4. Set working time. Click on “Time” in (Fig 2) to enter interface as (Fig 3) to set work time. The system default work time is 30 minutes. Click on “<<” and “>>” to regulate, the range is 1 to 120 minutes.

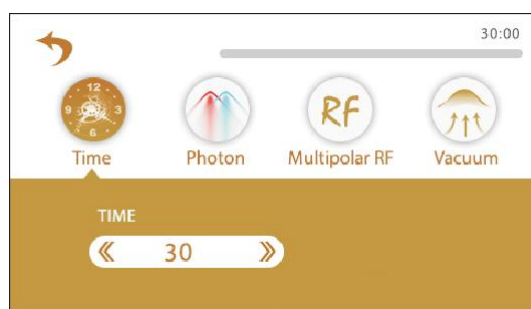

(Fig 3)


5. Set photon color and mode. Click on “Photon” to enter interface as (Fig 4).

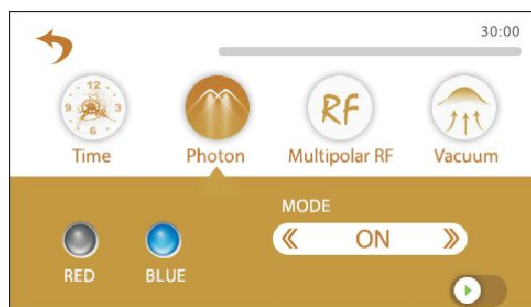

(Fig 4)

- (1) The system default photon color is blue. Click on “RED”, to change to red.
- (2) The system default photon mode is “ON” state, Click on “<<” and “>>” to set. “OFF” is turn off the photon function. “SY” indicate photon mode and vacuum mode are synchronous.

6. Set Multipolar RF size and intensity. Click on “Multipolar RF” to enter interface as (Fig 5) to set RF parameters.

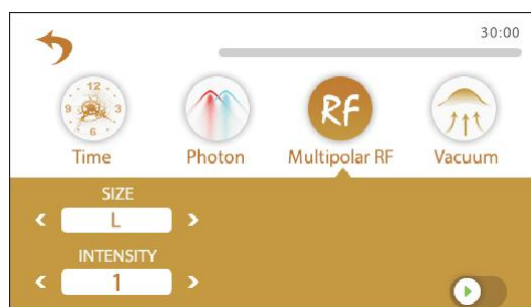


(Fig 5)

- (1) The system default is “L” is Multi-RF photon Vacuum liposuction head. According to the actual insert work head to select. Click on “<” and “>” to choose. “M” indicate middle multi-RF photon Vacuum liposuction head. “S” indicate small multi-RF photon Vacuum liposuction head.
- (2) The system default RF intensity is 1. Click on “<” and “>” to set, the range is 1(weak) to 10(strong).

7. Set vacuum parameter. Click on “Vacuum” to enter interface as (Fig 6) shows.

(1)Set vacuum mode. The system default vacuum mode is P1, click on “<” and “>” to adjust, The details refer to following table.

Mode	P1	P2	P3	P4	P5	P6
Inspiration	0.1S	0.2S	0.4S	1.0S	2.0S	--
Deflation	0.1S	0.3S	0.6S	0.8S	1.0S	--

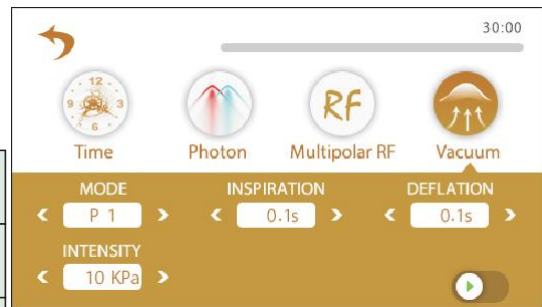

(Fig 6)

NOTE: P1~P6 modes are system default. If they don't fit for you, please click on “<” and “>” of “INSPIRATION” and “DEFLATION” regulate. The Inspiration/Deflation time is 0.1S to 3.0S. P1 to P3 mode can be adjusted, P4 to P5 mode can not be adjusted. P6 mode is only Inspiration mode, can't regulate time.

(2)Set vacuum intensity. The system default vacuum intensity is 10KPa, click on “<” and “>” to adjust, the range is 10KPa(weak) to 80KPa(strong). The beautician should consider the nursing part and make sure the guests can accept the pressure.

8.Click on “▶”, the system begin to work, click on “||” to stop working. Actually vacuum and actually temp. stands for the real data while the head working.

9. Beautician hold the Multi-RF photon Vacuum liposuction handle cling to the skin, and then move slowly. The action should be coherent.

10.The system will pause automatically if the setting time is over. Click on any key to continue.

11.Clean the leftovers with hot towel, use warm towel to clean the work head.

Cautions:

1.Should scribble smearing oil on the skin before doing eliminate fat treatment, which is good to smoothly moving, reduce the pain and avoid injure the skin.

2.Adjust the vacuum intensity should according to the acceptable of the guests, should from weak to strong, otherwise flaccid skin prone to congestion.

3.The operation should in muscle or fat fullness, the enrich subcutaneous parts. it is inappropriate for petite and rugged, more hair parts.

4.There are redness due to the personal and health status, which is a normal phenomenon that can be dissipated on its own.

5.If the filter cup is half full, please clean it immediately to avoid damage caused by flow backwards. It may be causing mechanical damage. Cleaning method: Pull the filter cup out from the host. Then pull the filter cup cap down, remove the clean impurities. Pay attention to the inside sealing ring. If leaking air, please check or replace the sealing ring.

Forbidden Group

1. People with heart disease or high blood pressure, or configured cardiac pacemaker.
2. Patients with acute inflammation, asthma, deep vein thrombosis, thyroncus, cancer.
3. People with hemorrhagic disease, trauma or who is bleeding.
4. Pregnant women
5. Medical Plastic parts, or parts with artificial in filling inside.
6. People with metal inside the body(include metallic tooth)
7. Patient with skin inflammation or with edema.
8. People with an abnormal immune system.
9. Numb or insensitive to heat.

Technical Parameters


Input Voltage: AC220~230V 50/60Hz

Output Power: $\leq 90VA$






Fuse: FL5X20\3.15A with buffer(double fuse)

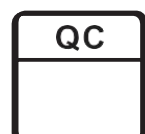Packing List

1. Host	1SET	<input type="checkbox"/>
2. Multi-RF photon Vacuum liposuction handle connect table	1PC	<input type="checkbox"/>
3. Large multi-RF photon Vacuum liposuction head	1PCS	<input type="checkbox"/>
4. Middle multi-RF photon Vacuum liposuction head	1PCS	<input type="checkbox"/>
5. Small multi-RF photon Vacuum liposuction head	1PCS	<input type="checkbox"/>
6. Power supply line	1PC	<input type="checkbox"/>
7. User manual	1COPY	<input type="checkbox"/>
8. Accessories frame	1PC	<input type="checkbox"/>
9. Mushroom hexagonal screw(M4X16)	1PCS	<input type="checkbox"/>
10. Hexagonal socket wrench(M3)	1PC	<input type="checkbox"/>
11. Searing ring(5X1 black)	10PCS	<input type="checkbox"/>





Symbols and Meanings

Warning Symbol	Meaning
 Danger	If used incorrectly will bring the risk of death or serious injury!
 Warning	If used incorrectly will cause possibility ill of death or serious injury!
 Attention	If used incorrectly will cause personnel injury or damage to the goods!
 Danger	Device exists high pressure risk of high voltage output terminals!
 Warning	If used incorrectly will cause personnel injury!

*Things damage refers to housing, property, livestock and pet damage.

Contraindications

<p>1. The following symptoms were forbidden to use</p> <p>(1)Patients who used by Heart rate regulator, cardiac pacemaker, ECS;</p> <p>(2)The use of artificial bone; Patients with silicosis;</p> <p>(3)Drunk or high fever patients;</p> <p>(4)Patients with higher sensitivity mental disease and epilepsy patients</p> <p>(5)Patients with no independent expression consciousness and can not take care of themselves.</p> <p>(6)Patients with metal materials in body (including metal teeth), pregnant women, cachexia, malignant tumor.</p> <p>(7)Severe hypertension and severe patients with cardiovascular and cerebrovascular disease.</p> <p>(8)Patients with renal failure.</p> <p>(9)Patients with severe infection</p>	 Danger
<p>2. The following please use under the guidance of professionals</p> <p>(1)Women during menstruation;</p> <p>(2)Children;</p> <p>(3)Hemorrhagic disease or trauma, inflammation, skin disease, skin infection;</p> <p>(4)Patients with feeling numb, no response to cold and heat;</p> <p>(5)The immune system has defects, scar formation history, physical disease;</p> <p>(6)The people who has taken face-lifting or her body has artificial thing.</p>	 Warning

Symbols and Meanings

1. Appliances and medical equipment close to the machine may be resulting in operational failure due to interference.
2. Prohibit use of the equipment in the following environment
 - Hot and humid environment
 - Dusty environment
 - Environmental with much smoke or steam
3. If there comes a failure, please shut down the power immediately, and send it to the distributor or manufacturer for maintenance. Do not repair by yourself.

4. Exclusion of liability.

The Company is not responsible for the following circumstances:

- (1) If fire, earthquake, war occurred;
- (2) The fault of the user, operator error;
- (3) Damage caused by using other method (not included in the use manual).
- (4) Operate without the manual's guide.
5. Do not make the machine close to the wall, keeping a 30CM space around for cooling.

Safety and Environmental Considerations

Toxic and hazardous substances reference list Electronic information product pollution control and management measures Table of contents of toxic and hazardous substances or elements

Part Name	Toxic and Hazardous Substance or Element						Remark
	Lead (Pb)	Mercury (Hg)	Cadmium (Cd)	Hexavalent chromium (Cr(VI))	Polybrominated biphenyls (PBB)	Polybrominated diphenyl ethers (PBDE)	
Plastic Board	○	○	○	○	○	○	
Printed Circuit Board	×	○	○	○	○	○	
Power Supply	×	○	○	○	○	○	
Accessory	×	○	○	○	○	○	

Declaration:

- The content of the restricted substances in all homogeneous materials of this part is below the limits set by the SJ/T11363-2006 standard.
 - ×
- The content of a homogeneous material in the part of the material is beyond the SJ/T11363-2006 standard requirements. The user's normal use of this product within the time limit of environmental protection, these substances or elements will not occur leakage or sudden change, will not cause the personal, property damage. For such substances or elements, the user shall not dispose of it yourself, please according to government regulations to the relevant departments of the government designated recycling.